# Bionect® Control Spray

Acido ialuronico sale sodico  
Argento metallico  
Vitamina E

## Indicazioni

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Acido ialuronico sale sodico  
Argento metallico  
Vitamina E

## Indications

Treatment of non infected skin lesions (e.g. abrasions, grazes, cracked skin, cuts, small surgical wounds, localized first and second degree burns), in diabetic foot, athlete's foot and pressure sores.

## How it acts

BIONECT CONTROL SPRAY is a suspension spray for topical use containing hyaluronic acid (HA), metallic silver and vitamin E. The hyaluronic acid and silver are combined in a matrix (kaolin and starch) with absorbent properties.

Absorption of the exudate creates an environment that facilitates the natural tissue regenerating process, forming, together with the HA and silver, a protective barrier against microbial penetration. The film-forming action of vitamin E, by assisting the re-epithelialization process, favours restoration of the optimal skin moistening conditions.

## Dose and method of use

- 1) Wash the lesions before applying the spray.
- 2) Shake the container well for a few seconds before use.
- 3) Holding the container in a vertical position, spray a quantity of powder sufficient to cover the surface to be treated from a distance of about 15 cm.
- 4) We recommend the medicated part be covered with sterile gauze.
- 5) Apply once or twice a day, according to the clinical condition.
- 6) The product is easily removed by washing the lesion with saline

## Warnings

*External use: do not swallow or inhale. Avoid contact with the eyes. If contact occurs, rinse with plenty of water. Avoid exposing the treated area to the sunlight. Avoid direct contact of the spray dispenser with the wound. In case of adverse reaction, discontinue the use of the product and consult a doctor. Consult a doctor for prolonged use. Close the package after use. Do not use if the package is damaged or after the expiry date*

## Composition

Principal component: 0.25% hyaluronic acid sodium salt  
Other components: metallic silver, kaolin, corn starch, vitamin E, rice bran oil, terpineol, disiloxane and propellant.

## Contraindications

Known hypersensitivity to any of the components. No overdose symptoms are known. No interactions with drugs or medicinal products are known. Pregnant women should consult a doctor before using the product.

## Storage

Store at a temperature between +5°C and +30°C

Danger: Extremely flammable aerosol. Pressurized container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children.

The lot no. and expiry date are indicated on the bottom of the container. The expiry date refers to the undamaged product correctly stored. Contains no ozone-depleting propellant gases. After use, dispose of in accordance with the local regulations in force.

## Manufacturer

Nirial Pharma srl, Via Rovani, 7  
20123 Milano (Italy)  
www.nirialpharma.com

## Distributed by:

Fidia Farmaceutici S.p.A., Via Ponte della Fabbrica, 3/A 35031 Abano Terme (PD) Italia

Date of last revision of the instructions for use: September 2021

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Iromedica AG - Oberstrasse 222 - CH-9014 St. Gallen